

Tetric EvoFlow® Bulk Fill / Tetric EvoCeram® Bulk Fill


1

Self-Etch




or

Etch & Rinse

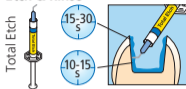

or

Selective Enamel-Etch

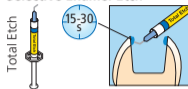

2

Adhese® Universal
VivaPen®



or

Adhese® Universal
Bottle



3

Tetric EvoFlow®
Bulk Fill



4

Tetric EvoCeram®
Bulk Fill



$\geq 500 \text{ mW/cm}^2$	20 s
$\geq 1'000 \text{ mW/cm}^2$	10 s
Bluephase® Style	10 s

Bluephase® Style







ivoclar
vivadent
clinical



**Tetric EvoFlow® Bulk Fill /
Tetric EvoCeram® Bulk Fill**
The bulk-fill composites

CE 0123

	Conventional composites				Bulk Fill composites	
	FLOWABLE		SCULPTABLE		FLOWABLE	SCULPTABLE
	Tetric EvoFlow®		Tetric EvoCeram®		Tetric EvoFlow® Bulk Fill	Tetric EvoCeram® Bulk Fill
						
	Dentin & Bleach XL	Enamel	Dentin & Bleach XL	Enamel	Dentin-like translucency	Enamel-like translucency
Layer thickness up to	1.5 mm	2 mm	1.5 mm	2 mm	4 mm	4 mm
Exposure Time ≥ 500 mW/cm ²	20 s					
≥ 1000 mW/cm ² or Bluephase® Style (1'100 mW/cm ²)	10 s					

**ivoclar
vivadent**
clinical

Date information prepared: 2015-05-11/63 9248/ Rev. 2
Ivoclar Vivadent AG, FL-9494 Schaan/Liechtenstein

see
Instructions

