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**Stand der Information:** 11-2015

**Procedure:** Manual and mechanical procedure for processing re-sterilisable medical devices

**Products:** Concorde II, Adapter, Double Adapter

**Instructions:** Due to the product design and the materials used, no definite limit to the maximum number of performable processing cycles can be specified. The service life of the medical devices is determined by their function and careful handling.

---

**Preparation procedure at the site of application:** Remove general soiling from the products directly after application. Do not use fixing agents or hot water ( $> 40^{\circ}\text{C}$ ), as this causes fixation of residues and can impair successful cleaning. In order to avoid contamination drying on, soak the used products in a disinfectant bath.

**Transport:** Safe storage in a closed container and transport of the products to the processing location, in order to avoid damage to the products and environmental contamination.

**Preparation for decontamination:** No particular requirements.

**Pre-cleaning:** No particular requirements.

**Manual cleaning and disinfection in the ultrasound apparatus:** For cleaning the products in the ultrasound apparatus, put the products in a beaker filled with the cleaning liquid, place in the ultrasound apparatus sufficiently filled with water and 2% contact liquid and start the ultrasound cleaning process. ❶ 30min primary cleaning at  $25^{\circ}\text{C}$ , stage 5 with 2% cleaning agent concentration. ❷ Generous manual rinsing under running water (reverse osmosis water). ❸ Dry using compressed air.

**Mechanical cleaning and disinfection in the washer/disinfector:**

Place products in the cleaning and disinfection equipment using the specified cleaning rack and start the cleaning process. ❶ Pre-cleaning with 10l cold water. ❷ 10min primary cleaning at 55°C (10.5l water and 62ml cleaning agent (DOS 1)). ❸ Rinse with 9.0l cold water and 13ml cleaning agent (DOS 3). ❹ Rinse with 9.0l cold reverse osmosis water. ❺ Thermal disinfection for 5 min at 90-93°C with 9.5l reverse osmosis water. ❻ Dry for 35 min at 99°C.

**Functional testing, maintenance:** Visual inspection for cleanliness. If necessary, repeat the reprocessing procedure until the product is visually clean.

**Packaging:** Standardized packaging of products for sterilisation according to ISO 11607 and EN 868.

**Sterilisation:** Steam sterilization of products, observing the respective national requirements. 3 pre-vacuum phases: Heat up to a sterilization temperature of 134°C; Shortest hold time: 3 min; Drying time: at least 20 min.

**Storage:** Store sterilised products in a dry, clean, and dust-free environment at moderate temperatures of 5°C to 40°C.

**Information on the validation of reprocessing:** The following test instructions, materials and machines were used during the validation:

Cleaning agents: Tickopur TR 13 (contact fluid), Dr. H. Stamm GmbH; Stammopur DR 8 (disinfectant cleaner), Dr. H. Stamm GmbH; Neodisher Mediclean forte, Dr. Weigert Co. (dosing system DOS 1); Neodisher Z (neutralisation agent), Dr. Weigert Co. (dosing system DOS 3); SonoCheck (BAG Health Care) (indicator for ultrasound effectiveness).

Cleaning/Sterilization devices: RDG: cleaning and disinfecting apparatus: Miele G7892 CD; Powersonic® P 2600 D ultrasound cleaning apparatus (Martin Walter Ultraschalltechnik AG); Autoclave Systec VX-95 (Systec GmbH).

Item carriers for washing: Upper basket/injector O177 / 1.

**Additional instructions:** If the above-mentioned chemicals and machines are not available, the user is obliged to validate his procedure accordingly.

---

It is the obligation of the user to ensure that the reprocessing procedure, including resources, materials and personnel, is suitable to achieve the required results. State-of-the-art technology and national laws require the compliance with validated processes.

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**Coltene/Whaledent GmbH + Co. KG**